
KOOL-DAM™

Heatless Liquid Dam and Block Out Resin Light Cure

KIT:  2 x 3 mL syringes Kool-Dam

BULK PACK:  10 x 3 mL syringes Kool-Dam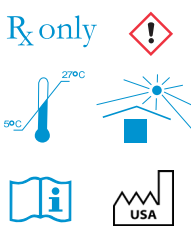

Kool-Dam has been uniquely formulated to eliminate the problems associated with paint on dam materials.

- Does not produce heat when cured. Low exothermic reaction eliminates burning and pain assuring patient comfort.
- Does not run. A single application is light cured one time only. Eliminates multiple passes and multiple light curing procedures.
- Remains flexible after curing. Truly a rubber dam substitute, the rubbery cured material is easily removed and is tear resistant.

PRODUCT DESCRIPTION

Light cure heatless liquid dam and block out resin

INDICATIONS

Power bleaching: For protection of the gingiva during power bleaching procedures.

Block-out resin: To block out undercuts and gingival embrasures prior to taking impressions.

Sandblasting: To protect the gingiva and mask areas of teeth during sandblasting procedures.

CONTRAINDICATIONS

Does not replace rubber dam for endodontic procedures

DIRECTIONS FOR USE

1. Place a green 18-gauge or pink 20-gauge applicator tip securely on the luer lock syringe.
2. Apply a coat of Kool-Dam 1-2 mm thick on the gingiva or tooth surfaces to be protected prior to power bleaching, sandblasting or other procedures requiring intraoral protection, or place in areas to be blocked out prior to taking impressions. Kool-Dam does not run and can be placed over multiple teeth before curing. Re-cap immediately after use.
3. Light cure for 10 seconds, depending on the light you are using.

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.