4 x 1.5gm Flows-Rite

Rx only

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Flows-Rite™

Multi-Purpose Flowable Composite

Flows-Rite is a 70% filled, light cured flowable composite, veneer cement, liner, base and fissure sealant with the handling characteristics and esthetic properties most preferred by dentists. It is radiopaque, fluoride releasing and has a nominal particle size of 0.7-3.0 microns. Flows-Rite is available in the following shades: A1, A2, A3, A3.5, and ShadeFusion. ShadeFusion is a unique chameleon shade. It blends shades and reflects adjacent hues to create a natural esthetic, and it is ideal for veneer cementation.

PRODUCT DESCRIPTION

Multi-purpose flowable composite, veneer cement, liner, base and fissure sealant.

INDICATIONS

Recommended as a filling material for pits, root surface cavities, and Class I, II, III, IV and V restorations where there is no pulpal involvement, and as a liner, base, fissure sealant, and veneer cement.

CONTRAINDICATIONS

Not for direct placement on the exposed pulp.

INSTRUCTIONS FOR USE

1. Isolate and prepare tooth to receive a restoration. Place appropriate pulp protection, if indicated.
2. Etch and apply a bonding agent following the manufacturer's instructions for use.
3. Place a pink 20-gauge applicator tip on the Flows-Rite syringe and carefully start flow of composite. Apply in 2mm increments light curing after each layer. Do not force the syringe. Use a new applicator tip for each patient.

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4. Finish and polish as usual.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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