Tray Adhesive

PRODUCT DESCRIPTION

Use Tray Adhesive to enhance the adhesion of impression materials to metal, plastic, and various other types of impression trays. Apply a thin layer of Tray Adhesive to the tray to keep impressions attached during removal from the mouth and capture the most accurate impressions.

INDICATIONS

For enhancing the adhesion of impression materials to different types of trays.

CONTRAINDICATIONS

Tray Adhesive does not have any contraindications when used within the stated indications. For any potential allergies, refer to associated MSDS.

XP-TRAY-IN-01w
REV: 09/2024

INSTRUCTIONS FOR USE

1. Apply a thin layer (barely visible) of tray adhesive to the impression surface of desired tray using the supplied applicator brush.
2. Allow to dry for at least 3 minutes, but no longer than 10 minutes.
3. Repeat application if more than 10 minutes have elapsed after initial application.
4. Lightly touch the tray with a gloved finger to test if it has properly dried prior to applying the impression material. It should feel tacky.
5. Load the tray with impression material and take impression.

CAUTION

- Tray Adhesive is a flammable liquid. Keep away from heat, sparks and all other ignition sources. temperature, contamination and sources of ignition.
- Flammable vapor may form if Tray Adhesive is exposed to air for long periods of time.
- Tray Adhesive may cause eye or skin irritation on contact. Prolonged inhalation over-exposure may cause mucous membrane irritation, fatigue, headache, loss of appetite.
- Always wear protective gloves and eyewear when using Tray Adhesive.
- If eye is exposed to Tray Adhesive, flush thoroughly with water at a sink or emergency eye wash station.

STORAGE AND HANDLING

- Keep Tray Adhesive stored tightly in original container at room temperature.
- Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 3 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to: Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262 / www.pulpdent.com /

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Ravelli S.p.A.
Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.