
## PREP-RITE™ RC

Viscous EDTA Gel, 15%  
Canal Lubricant and Chelating Agent

 4 x 5 gm Prep-Rite RC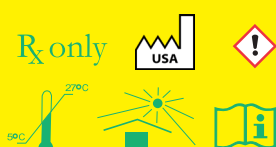

XP-PRC-IN-10w REV: 08/2024

Prep-Rite RC is a viscous 15% EDTA gel that has been buffered to a neutral pH. It contains peroxide, which provides an effervescent action, and has a viscous consistency that is easily picked up on a file. During instrumentation and debridement of the root canal, Prep-Rite RC is an effective calcium binding material for decalcifying the canal walls, and it acts as a lubricant to minimize the binding and breaking of files. Prep-Rite is water-based and easily rinses out of the canal.

**PRODUCT DESCRIPTION**

15% EDTA viscous gel root canal lubricant and chelating agent with peroxide

**INDICATIONS**

To assist with instrumentation and debridement during endodontic treatment

**CONTRAINDICATIONS**

Not a permanent root canal filling material

**DIRECTIONS FOR USE**

1. Always place a rubber dam.
2. Remove cap from the syringe.
3. Dispense Prep-Rite RC onto a pad. Recap syringe immediately after use.
4. Dip the file into the Prep-Rite gel. Prep-Rite picks up on the file.
5. Instrument the canal and irrigate in the usual manner. Reapply Prep-Rite RC and repeat as necessary until the canal is enlarged and shaped as desired.
6. Rinse canal thoroughly.

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

**CAUTION**

Avoid contact with eyes.

**STORAGE AND HANDLING**

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

**REPORTING OF SERIOUS INCIDENTS**

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation  
80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.