Porcelain Prep Kit

PULPDENT®

XK-PPK-IN-18w
REV: 07/2024

PRODUCT DESCRIPTION

The PULPDENT Porcelain Prep Kit contains the following materials for preparing porcelain surfaces for bonding: Porcelain Etch Gel 9.6% hydrofluoric acid; Kool-Dam™ light cure liquid dam; Dry-Rite Drying Agent; and Silane Bond enhancer.

INDICATIONS

For preparing porcelain surfaces and hybrid composites prior to cementation or bonding with resins and resin cements.

CONTRAINDICATIONS

Do not place in contact with teeth and soft tissues.

INSTRUCTIONS FOR USE

1. Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
2. Place rubber dam to isolate operative site and protect soft tissue.
3. Securely attach a green 18-gauge applicator tip to the Kool-Dam syringe and apply a coat of Kool-Dam 1-2 mm thick on the gingiva or tooth surfaces requiring protection, if indicated.
4. Light cure for 10 seconds. Cures with all lights.
5. Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
6. Securely attach a 25-gauge applicator tip to the Porcelain Etch Gel syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Never force plunger. Use a new applicator tip for each patient.
7. Carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Cap syringe immediately after use.
8. Remove gel with vacuum tip and rinse treated surface with copious amounts of water. Dry porcelain surface with clean uncontaminated air.
9. Securely attach red dropper tip to the Dry-Rite syringe and apply Pulpdent Dry-Rite drying agent to treated surface. To promote rapid drying, use a gentle flow of clean, oil-free air if desired. Cap Dry-Rite immediately after use.
10. Securely attach red dropper tip to Silane syringe and apply Pulpdent Silane Bond Enhancer to the etched ceramic surface and allow to dry. Apply a gentle flow of clean, oil-free and moisture-free air, if desired. Cap syringe immediately after use.
11. Proceed with cementation or bonding with resins and resin cements.

CAUTION

Porcelain Etch Gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous. Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.

WARNING

If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body.

Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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PULPDENT® Corporation